


AGE-Reader Quick Installation Guide.



Read user manual for: intended use, safety instructions and full instructions.

- Warning:**
- Uninstall previous version of software first
 - Do not connect the AGE Reader until the software has been installed


1. Check whether the serial number on the AGE-Reader is the same as on the CD.

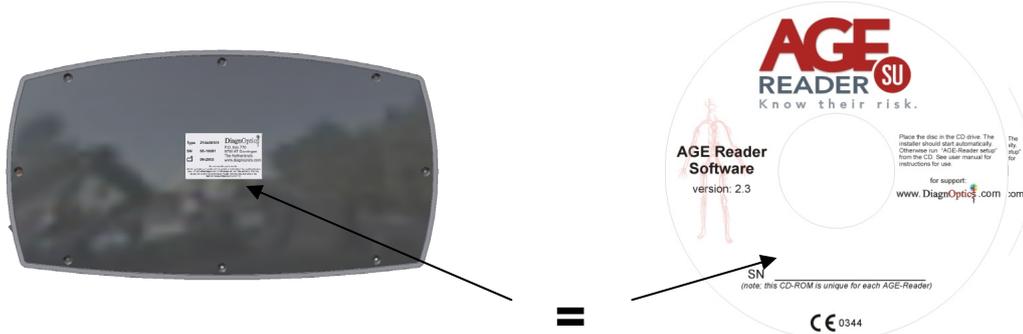

2. To install you need to have administrator rights. Insert the AGE-Reader CD in the CD/DVD-drive of a computer running 32-bit Windows 2000 or later. The installer should start automatically, otherwise run "setup23-eng.exe" from CD.




3

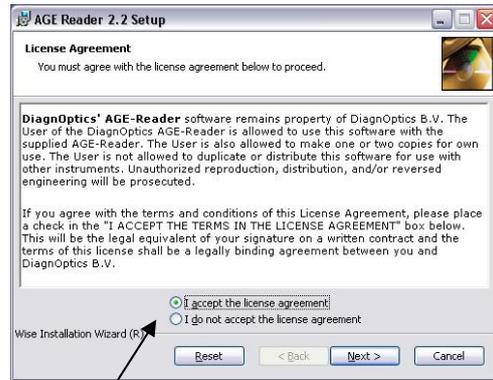

4 Read carefully before agreeing !


5

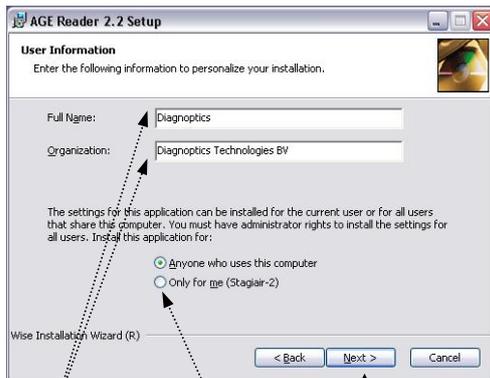

? Change the name of the user & company.


? Choose who has permission to use the AGE-Reader.

6

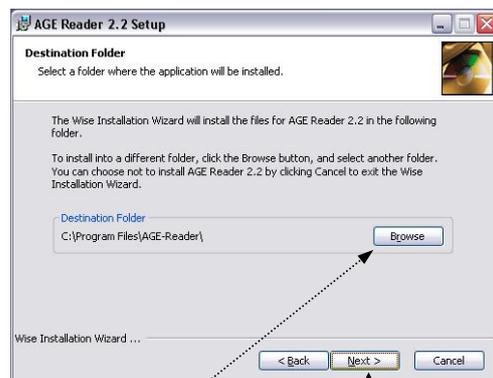


? Option to change install location.

7

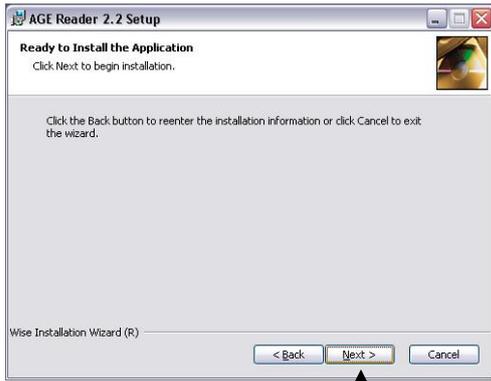

8

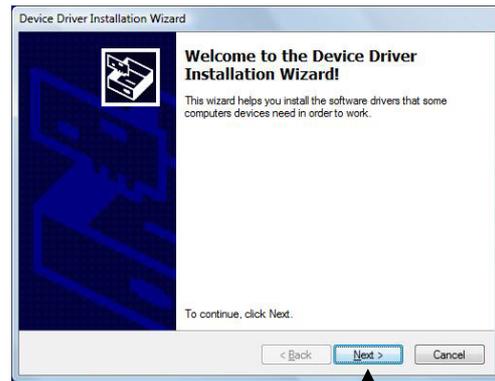

9



10



11

The software is now installed. The next steps will setup the hardware.

Hardware Installation



12 USB Connector;
Connect PC-side first.

13 Power Connector;
Connect mains-plug first.

14 On/Off Switch;
Turn it in the I position.

After turning the power on (step 14), Windows should detect the hardware automatically and will display the "Found New Hardware" dialog. Select 'do not connect to the internet' and press next. Select 'install software automatically', the software starts installing the driver. Please wait until windows reports the device to be ready for use.

Under Windows XP and VISTA a window may be displayed that the spectrometer board "has not passed Windows Logo testing to verify compatibility with Windows XP": Select "Continue Anyway".

For Windows 2000 The next dialog allows you to browse to the directory where the required USB driver is located. Browse to the Windows ...\\SYSTEM32\\DRIVERS folder and select the AvsUsb.sys driver.

The software can be started by the hyperlink on the desktop or from the 'start menu'.
On first use the user needs to determine the folder where the measurement data will be stored.

Uninstalling the AGE Reader Software

Removing the program by using: "Start\\Control Panel\\Software\\DiagnOptics" will remove all the files.

For Support: Visit the DiagnOptics website at: www.diagnoptics.com or www.age-reader.com